

Navina Classic Product information

Intended use

The Navina Classic is intended for transanal irrigation by instilling water up into the lower part of the colon through a rectal catheter.

Indications for use

Navina Classic is indicated to help adults and children from 3 years who suffer from fecal incontinence, chronic constipation and/or time-consuming bowel management. By instilling water up into the lower part of the colon, the peristaltic muscles in the bowel can be triggered and start to evacuate the content of the lower colon and rectum.

- Always consult a healthcare professional specialized in TAI before using Navina Systems.
- Only use Navina Systems for its intended use, as described in this instruction manual.
- Navina Systems is for a single user and should not be shared with other people.
- The Navina catheter regular is for adult use only.

Contraindications

You should not use Navina™ Classic system if you have one or more of the following:

- Known anal or colorectal stenosis
- Active inflammatory bowel disease.
- Acute diverticulitis.
- Colorectal cancer.
- Ischemic colitis.
- Within three months of anal or colorectal surgery.
- You are within 4 weeks of previous endoscopic polypectomy.

As the list of contraindications may not be exhaustive, healthcare professionals will always consider individual user factors as well.

Navina Classic Systems

Reference	Type	Specification
69005	Navina Classic System Regular For adult use only	1 Navina Classic control unit 1 Water container 1 Tube set 2 Rectal catheters Regular 1 Navina case 1 Accessory set
69007	Navina Classic System Small From 3 years	1 Navina Classic control unit 1 Water container 1 Tube set 2 Rectal catheters Small 1 Navina case 1 Accessory set
69017	Navina Classic System Cone From 3 years	1 Navina Classic control unit 1 Water container 1 Tube set 2 Cones 1 Navina case 1 Accessory set



Navina Classic System Consumables

Reference	Type	Specification	Technical details
69006	Navina Classic System Regular refill	1 Navina Classic control unit 1 Water container 1 Tube set 2 Navina catheters Regular*	100 uses 15 uses, 1.5 liter, foldable, double scale for visibility. 100 uses Hydrophilic, single use catheters.
69008	Navina Classic System Small refill	1 Navina Classic control unit 1 Water container 1 Tube set 2 Navina catheters Small	100 uses 15 uses, 1.5 liter, foldable, double scale for visibility. 100 uses Hydrophilic, single use catheters.
69018	Navina Classic System Cone refill	1 Navina Classic control unit 1 Water container 1 Tube set 2 Cones	100 uses 15 uses, 1.5 liter, foldable, double scale for visibility. 100 uses Hydrophilic, single use cones.
69003	Consumable set Regular	1 Water container 15 Navina catheters Regular*	15 uses, 1.5 liter, foldable, double scale for visibility. Hydrophilic, single use catheters.
69004	Consumable set Small	1 Water container 15 Navina catheters Small	15 uses, 1.5 liter, foldable, double scale for visibility. Hydrophilic, single use catheters.
69016	Consumable set Cone	1 Water container 15 Cones	15 uses, 1.5 liter, foldable, double scale for visibility. Hydrophilic, single use cones.
68940	Rectal catheter set Regular*	10 catheters	Hydrophilic, single use catheters.
68943	Rectal catheter set Small	10 catheters	Hydrophilic, single use catheters.
69011	Tube set	1 Catheter/Cone tube 1 Water container tube	100 uses 100 uses
69012	Accessory set	2 Grip rings (1 vertical, 1 horizontal) 1 Lanyard 1 Positioning strap 1 Navina Smart positioning clip	
69013	Navina Classic control unit	1 Navina Classic control unit	100 uses

* For adult use only

All Navina Classic components should be disposed of as household waste.

wellspect.com



CE Rx Only



Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden. Phone: +46 31 376 40 00.