

Navina Insert

Description

Navina Insert is a soft, flexible, liquid-filled insert designed to prevent fecal incontinence. The device is non-sterile and for single use only.

Indications for use

Navina Insert is indicated for the management of Accidental Bowel Leakage (ABL) due to bowel incontinence. The rectal insert is designed for self-insertion to seal and help prevent the involuntary leakage of stool from the rectum.

Risks

- Possible mild irritation or discomfort of the anus or anal canal
- Possible migration or displacement of the insert into the anal canal or rectum
- Unexpected expulsion

Warnings, Precautions, and Contraindications

Bowel leakage may be medically treated. Navina Insert is not intended to replace surgical options to treat accidental bowel leakage. Consult your physician for professional advice.

- Navina Insert is contraindicated for use in the pediatric population.
- This device is non-sterile and for single use only. Reuse of the device could result in device breakage.
- Do not use if the package is open or damaged.
- Do not use if you have a silicone allergy.
- Do not use with suppositories.

Discontinue use of Navina Insert and consult your physician if you have:

- Bloody stool or bleeding from the rectum.
- Bleeding hemorrhoids.
- Anal/rectal pain, irritation, spasms, tearing, or fissure.
- Displacement of the Fecal Incontinence Insert into the anal canal or rectum which has not been expelled with the next bowel movement.
- Undergone rectal surgery within the last 4 weeks.

Navina insert

Reference	Type	Specification
6930400	Navina Insert, Standard	Box of 30 pieces
6930500	Navina Insert, Large	Box of 30 pieces
6930600	Navina Insert, Starter Pack	Box of 5 Standard and 5 Large pieces



At Wellspect we develop innovative continence care solutions that improve quality of life for people with bladder and bowel problems. We inspire our users to build self-confidence and independence as well as good health and well-being. We have been leading the industry for over 40 years with our product brands LoFric® and Navina™. We always aim to minimize the environmental impact of our products and passionately strive to become climate neutral. We work together with users and healthcare professionals to improve clinical outcome in a sustainable way, now and for the future.

Wellspect. A Real Difference.

For more information about our products and services, please visit [Wellspect.com](https://wellspect.com).

Join the conversation on LinkedIn, Twitter, Facebook and Instagram.

wellspect.com

Wellspect HealthCare, Aminogatan 1, P.O. Box 14, SE-431 21 Mölndal, Sweden. Phone: +46 31 376 40 00.

Navina fecal incontinence insert



Manufacturer:
Minnesota Medical Technologies
2446 Henry Road Northwest
Stewartville, MN 55976 USA
507-533-0366
www.mnmedtech.us

Importer:
Wellspect HealthCare
Aminogatan 1
P.O. Box 14, SE-431 21
Mölndal, Sweden.
www.wellspect.com

MPS Medical Product Service GmbH
Borngasse 20
Braunfels 35619
Germany